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DE

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**Bed Rail RFM with bag**

UK


**IMPORTANT:** Please ensure that this instruction manual is carefully read and understood before use. Please keep this manual for future reference when necessary. In case you have any question, please contact us.

**Purpose**

This bed rail is designed to provide a stable support and help people move from lying position to sitting position, assist people getting in and out of bed and offer an extra support when people stand beside the bed.

**Contents**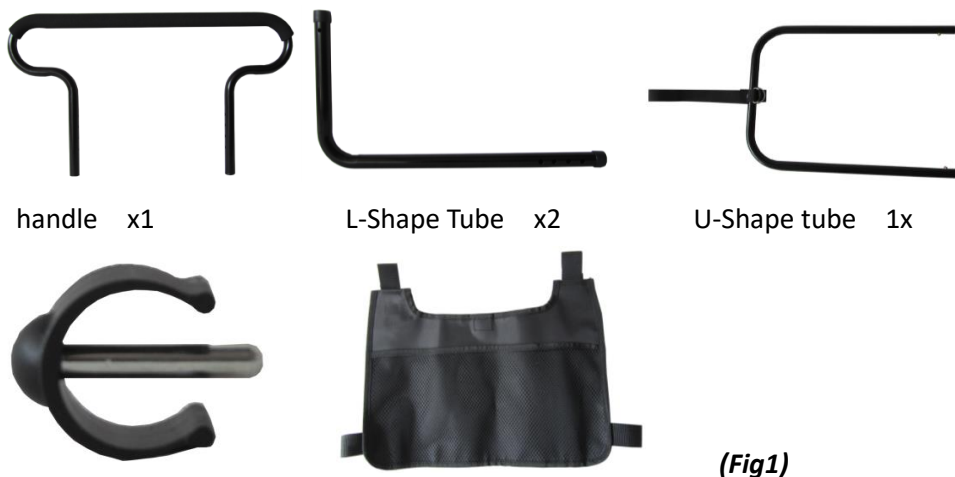

**(Fig1)**

### Installation instruction

1. Remove the protective polybags from the tubes.
2. Remove the mattress from the bed.
3. Fix the bag to the bed rail handle and fasten the bag by the Velcro (refer to **Fig 2**).

Fig 2

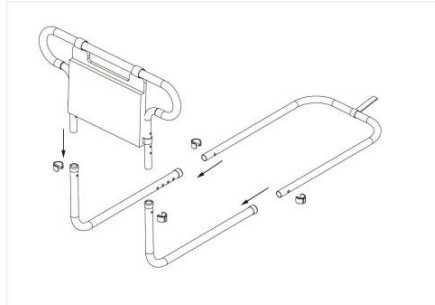
Fig 3


4. Insert the handle into the L-shape tube. Choose a suitable height(472-547 mm, 4 positions adjustable) and fix the tube by a C-clip(supplied in the package)(refer to **Fig 3**)

### Fig 3

5. Insert the U-shape tube into the L-Shape tube, choose a suitable length of the rail(872-947 mm, 4 positions adjustable) and fix the tube with a C-Clip (refer to **Fig 3**)

6. Fasten the strap to the bed with the D-ring (refer to **Fig 4**).
7. Replace the mattress.

### Fig 4

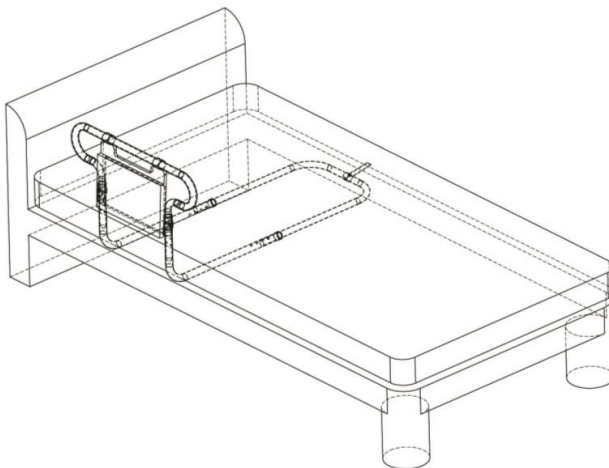### Bed rail positioning

1. There **MUST** be a gap of at least 250 mm from the bed headboard to the bed rail.
2. The bed rail should be placed in a suitable position that a user (who is lying in bed) is able to grab the bed rail without stretching.
3. A suitable position where the bed rail is fixed is between the user's neck and hip, so that the user can easily grab the bed rail and get off the bed.
4. When a bed rail is fixed beside the bed, the user who lies in bed should be aware that he lies/sleeps regularly, instead of moving too often from back to forth and from side to side, so as to reduce the risk of head/hand/hip/leg entrapment.

### Care and Maintenance

1. The bed rail should be checked as part of a planned maintenance program.
2. Stop using the bed rail immediately whenever there are signs of wear, damage, unstableness, the handle is loose, etc
3. Check that the bed rail is properly positioned and the strap fastened.
4. Check that both fixing C-clips are fitted correctly and secure.
5. Clean the surfaces of the bed rail with a pH-neutral cleaning agent. The manufacturer's instructions for use must be observed. Do not use any scouring agents, corrosive substances, acids, bleach or hard cleaning agents or high-pressure cleaners! Wipe disinfection should be carried out after cleaning with Surface disinfection. Ensure that the bed railing is completely wetted, do not wipe and allow it to dry for at least 30 seconds. Follow the instructions of the disinfectant manufacturer.
6. Disinfection is absolutely necessary before passing on the product to another user!
7. Do not scratch the surface with hard and sharp objects.

### Safety Instructions

1. This bed rail is designed for adults only, not for kids or children
2. This bed rail is designed for people whose weight is less than 150 kg (330LBS)
3. Always use the safety strap to fix the bed rail to the frame
4. Users must regularly check whether the bed rail is firmly fixed and stable enough
5. Make sure the mattress is firmly resting against the rails
6. Do not use this device to a bed that this bed rail is not designed for.

### Product Data:

Handle width: 624 mm

Rail width: 335 mm

Rail length: 872-947 mm, 4 adjustment(25.4 mm each)

Handle height: 472-547 mm, 4 adjustment(25.4 mm each)

Pouch(small) load: 0.5 kg

Pocket(big) load: 2 kg

Unit weight: 1.4 kg

Material: aluminium, EVA foam (ethylene vinyl acetate copolymer)

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Rehaforum Medical GmbH, Daimlerstraße 12a, DE-25337 Elmshorn