# CL7200

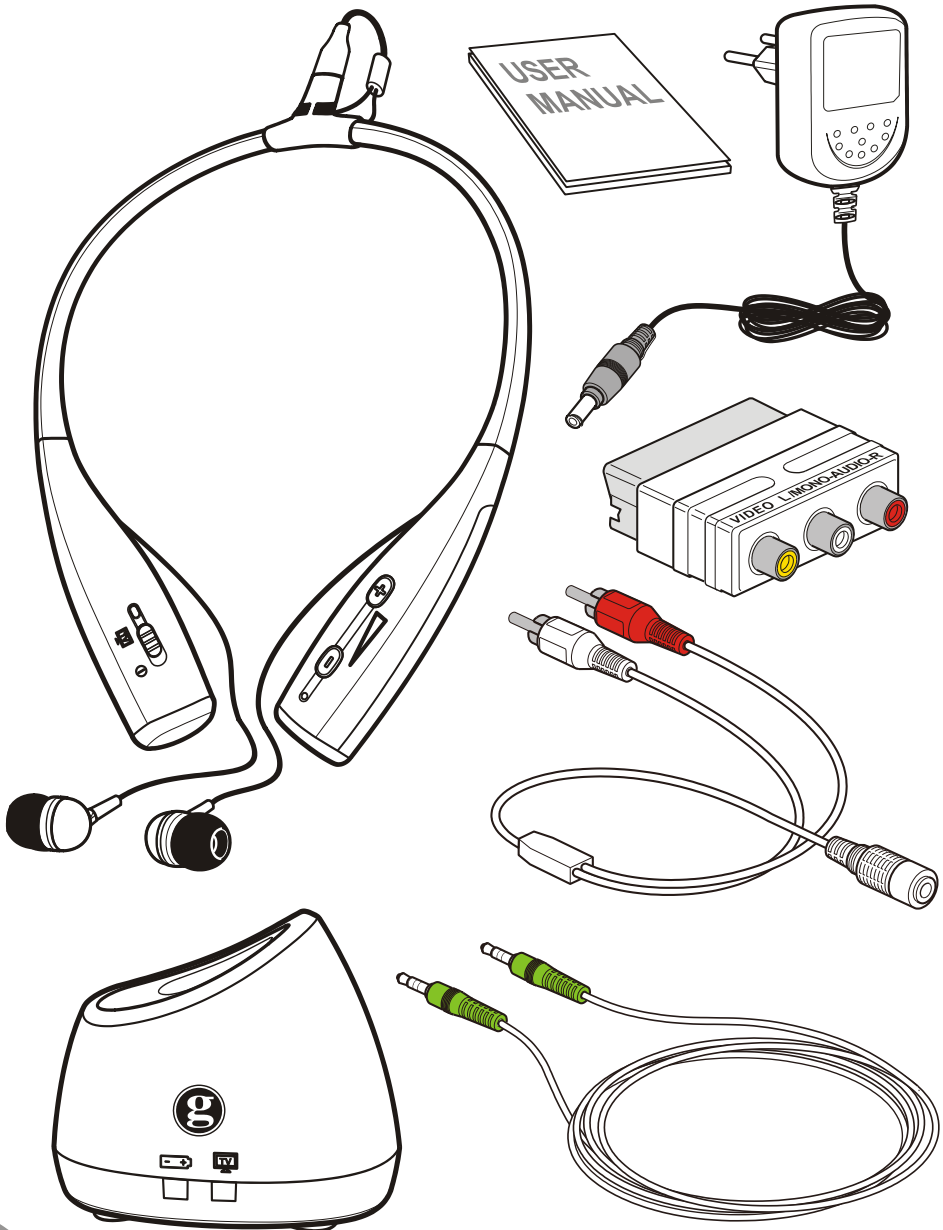
En	Digital TV Listener
Fr	Casque TV Numérique
D	Digitale TV-Kopfhörer
Español	Oyente activo de TV digital
Italiano	Cuffia per TV Digitale
Nederlands	Digitale TV Luisterapparaat
עברית	מאזין דיגיטלי לטלוויזיה
Svenska	Digital TV-lyssnare

En Unpacking  
Fr Contenu du Colis  
De Auspacken  
Esp Desempaque

It Estrarre dall'imballaggio  
NL Uitpakken  
עברית פריקה  
Svenska Packa upp


a


En Installation  
Fr Installation  
De Installation  
Esp Instalación

It Installazione  
NL Installatie  
עברית התקנה  
Svenska Installation

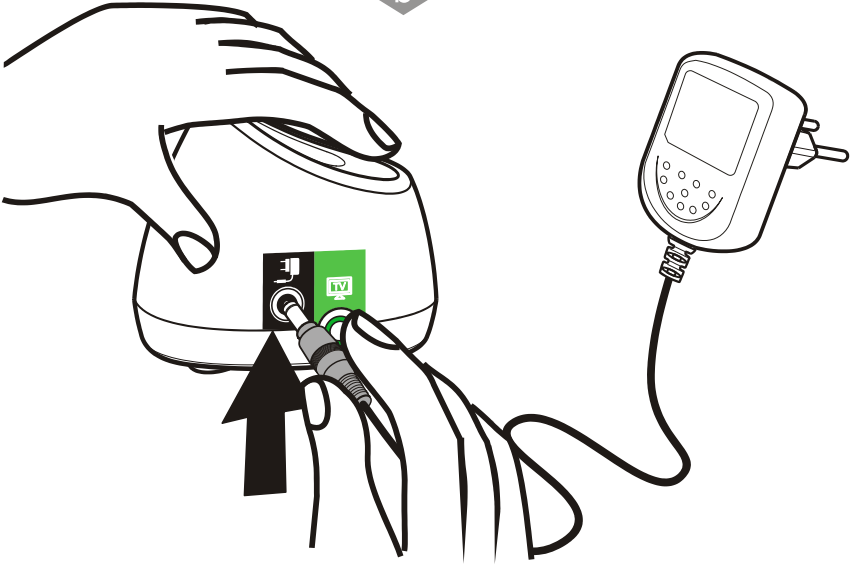b


c

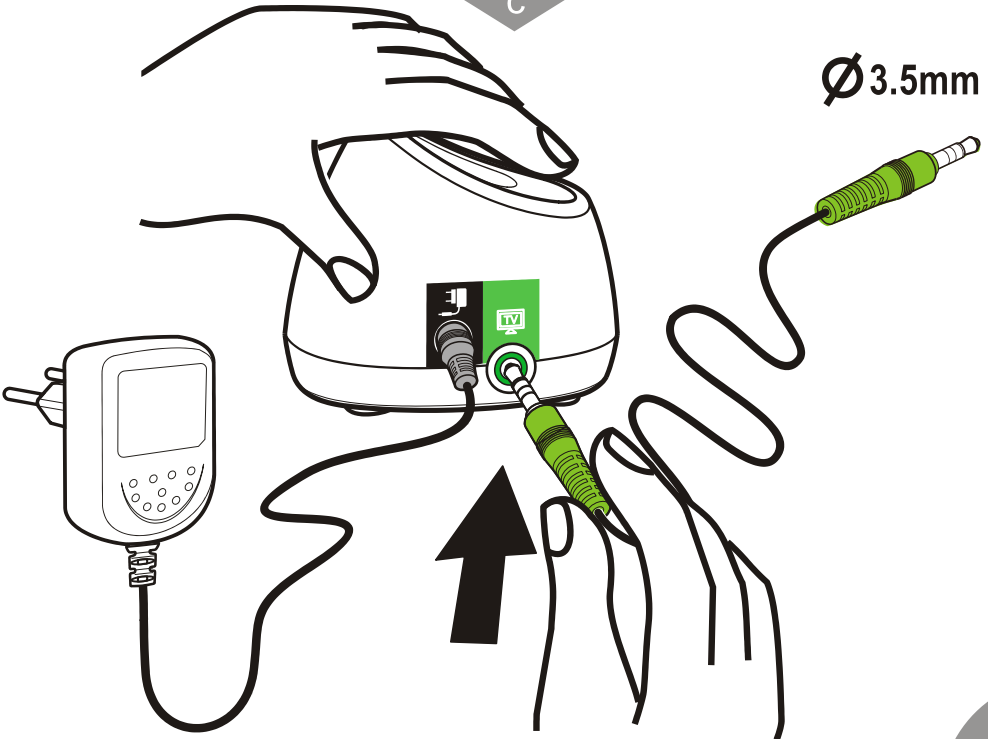
En	Installation	It	Installazione
Fr	Installation	NL	Installatie
De	Installation	עברית	התקנה
Esp	Instalación	Svenska	Installation

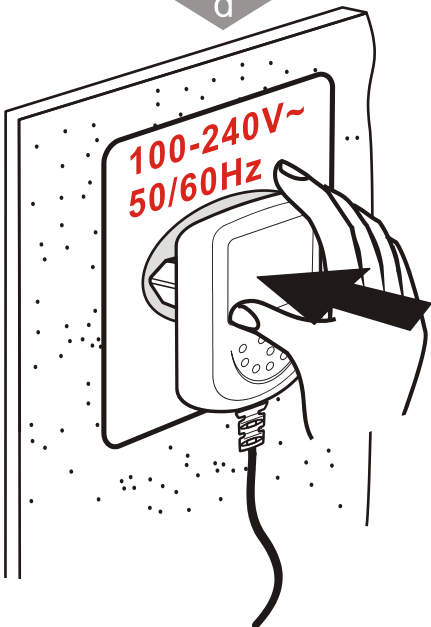
d

e

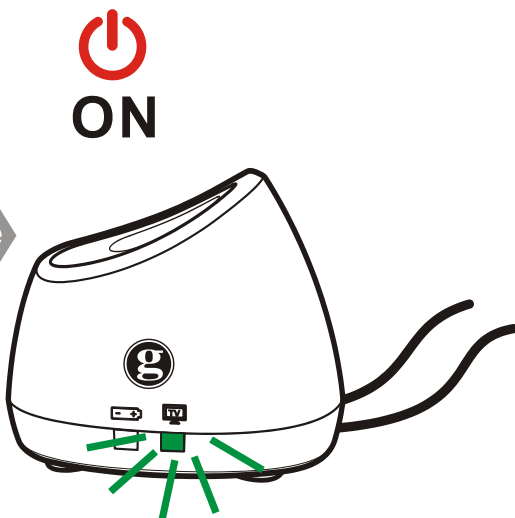
f

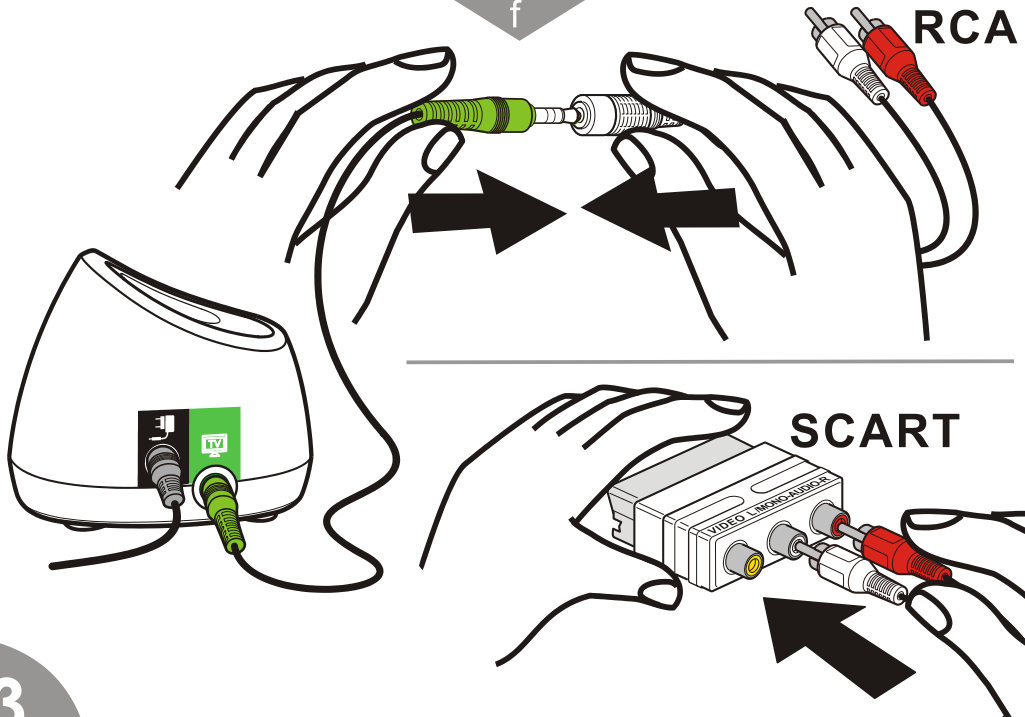
En Installation  
 Fr Installation  
 De Installation  
 Esp Instalación

It Installazione  
 NL Installatie  
 עברית התקנה  
 Svenska Installation

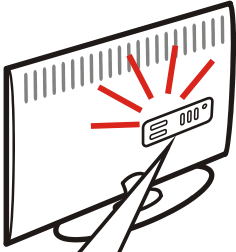g

TYPE A

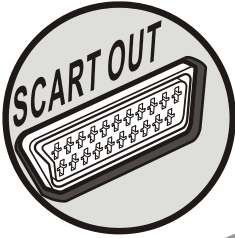
TYPE B


TYPE C

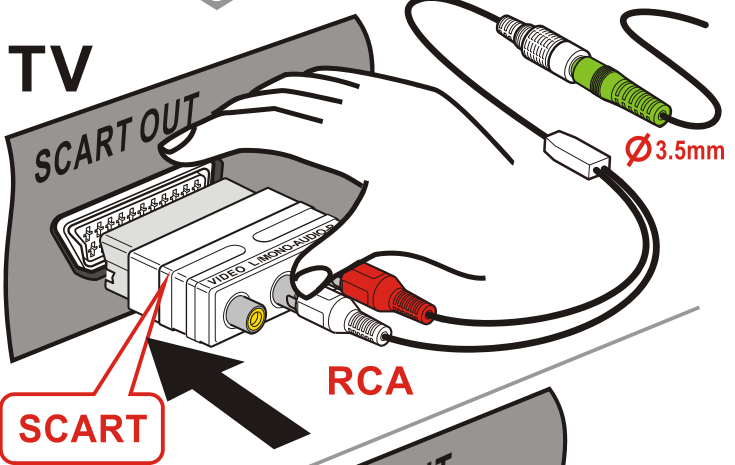
TV

TV

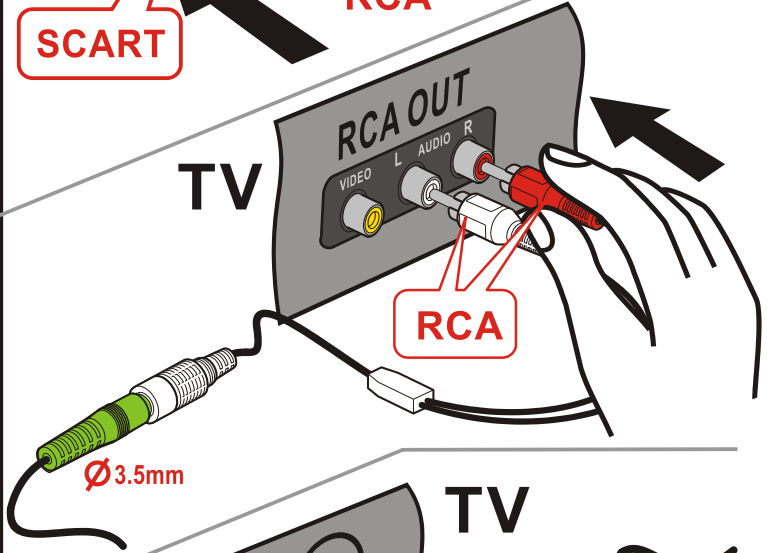
TV

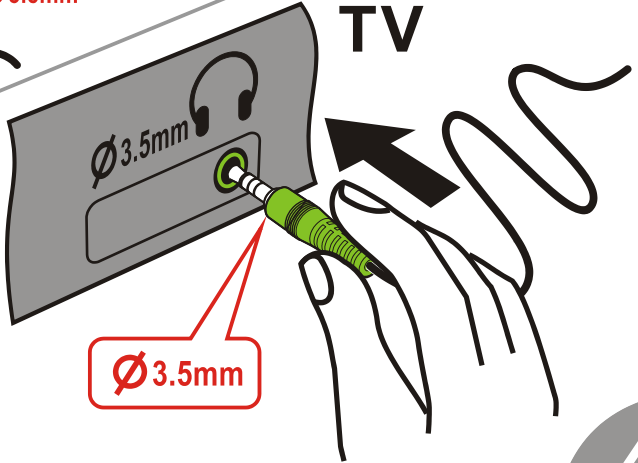
En	Charging	It	Ricarica
Fr	Chargement	NL	Opladen
De	Aufladen	עברית	טעינה
Esp	Carga	Svenska	Laddar

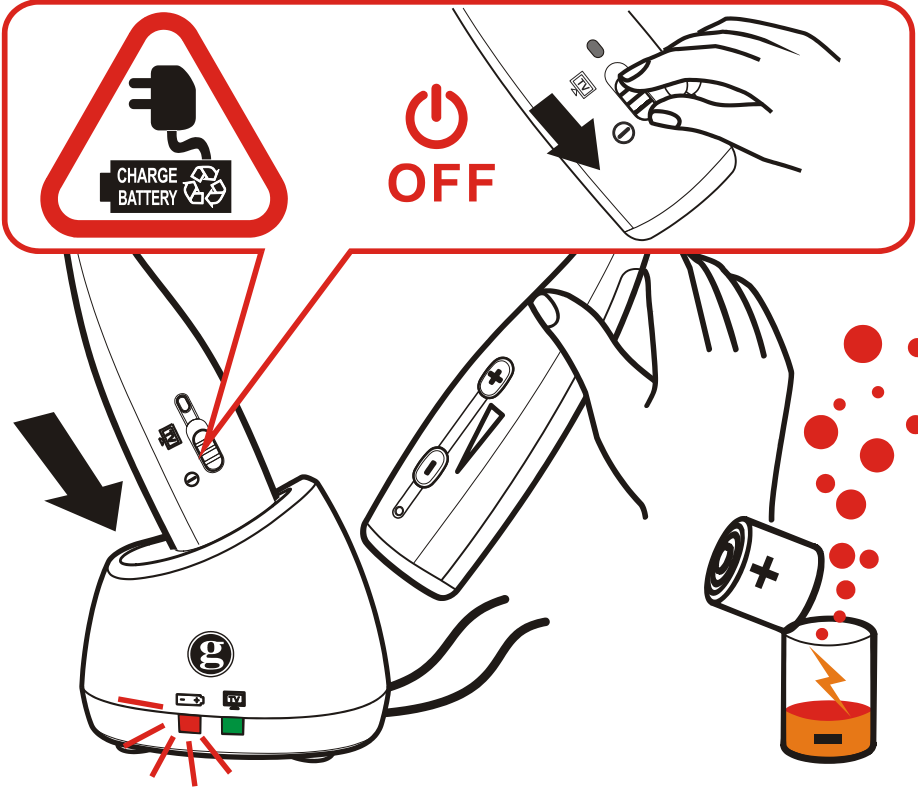h


i

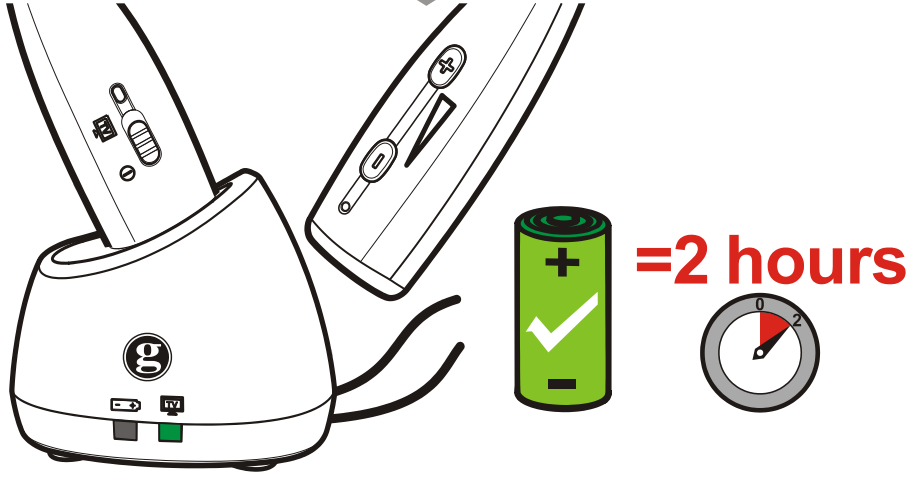En Pairing  
Fr Enregistrement  
De Anmelden  
Esp Emparejamiento

It Accoppiare  
NL Paarvorming  
עברית התאמה  
Svenska Parning

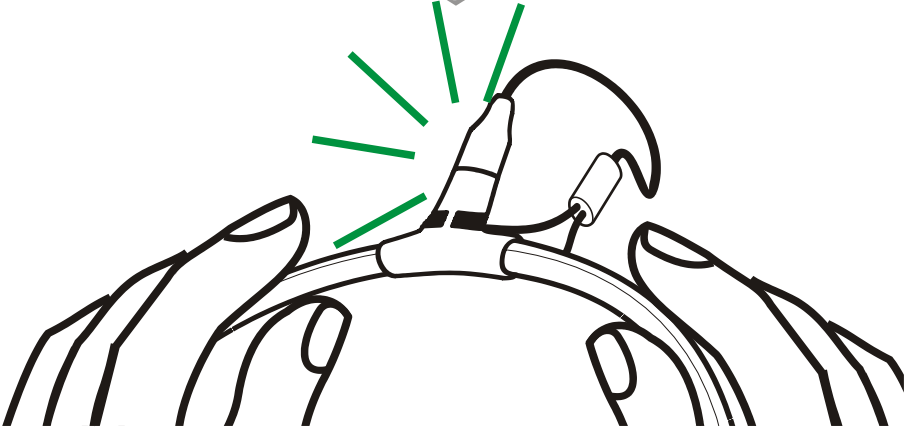
j


k

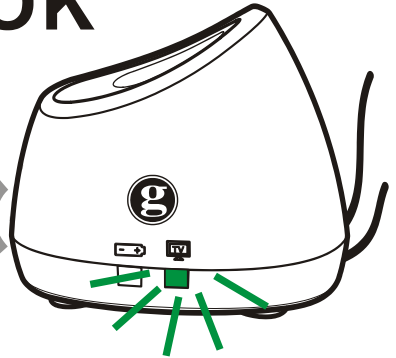ON → OK


En	Volume	It	Volume
Fr	Volume	NL	Volume
De	Lautstärke	עברית	עוצמת שמע
Esp	Volumen	Svenska	Volym

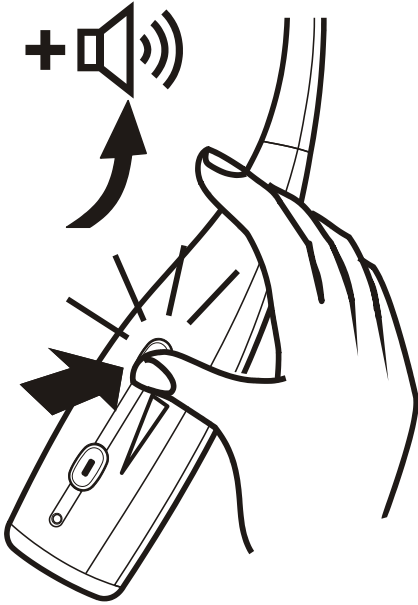**Max**

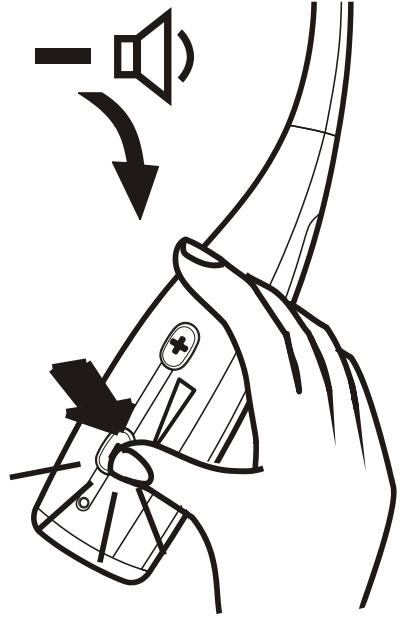**Beep~**

**Min**

**Beep~**

En	How to use
Fr	Utilisation
De	Bedienungsanleitung
Esp	Cómo se usa

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**En** : Your CL7200 is now set up to the recommended levels. For further personalisation (if required) see pages 11 & 12.

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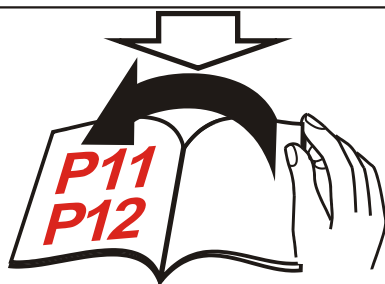
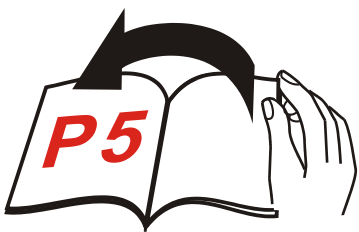
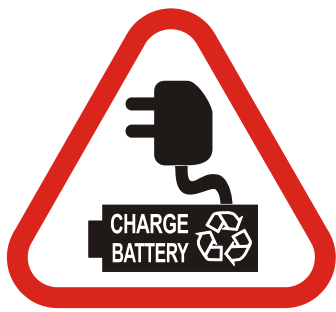
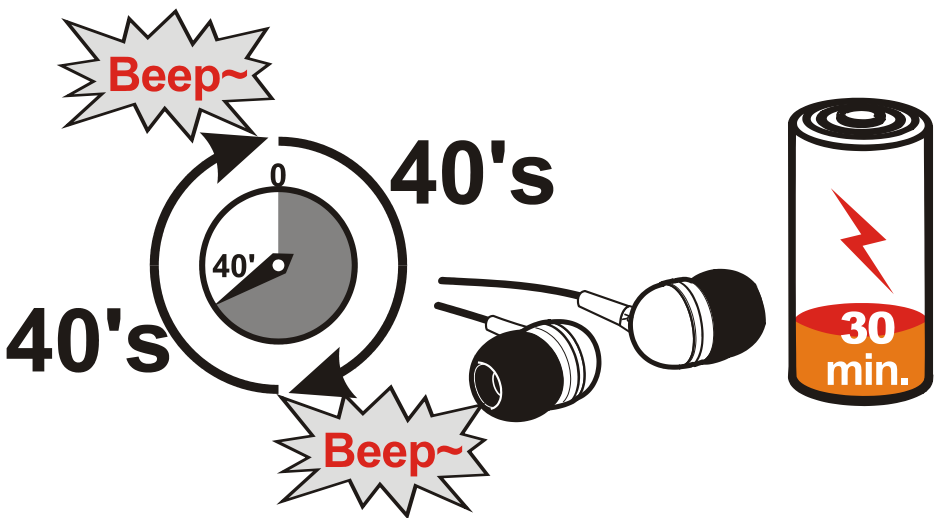
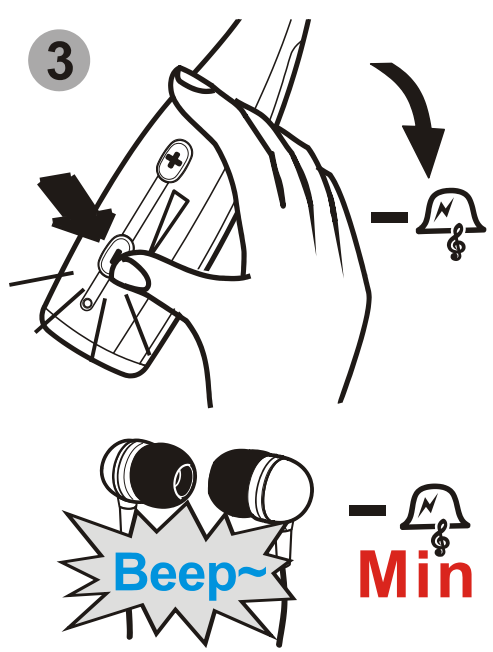
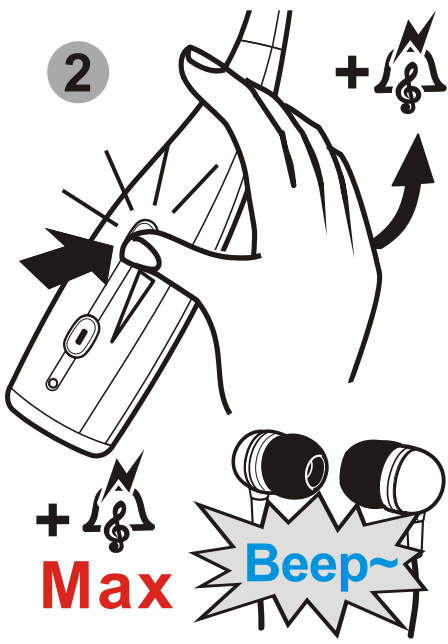
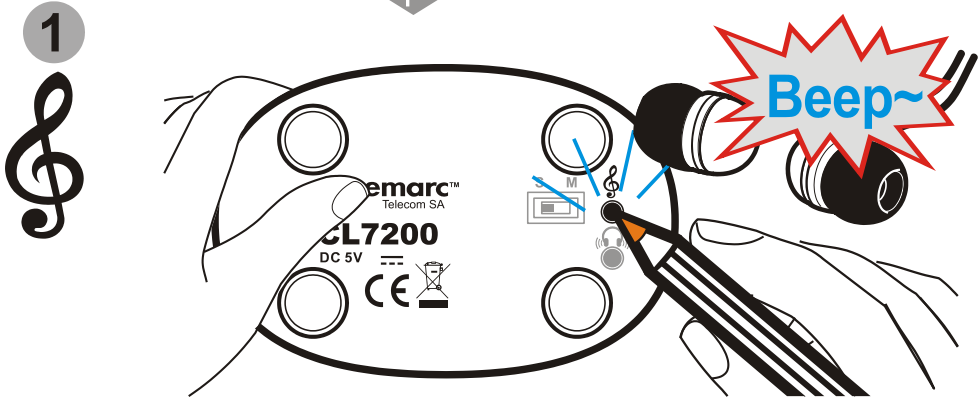
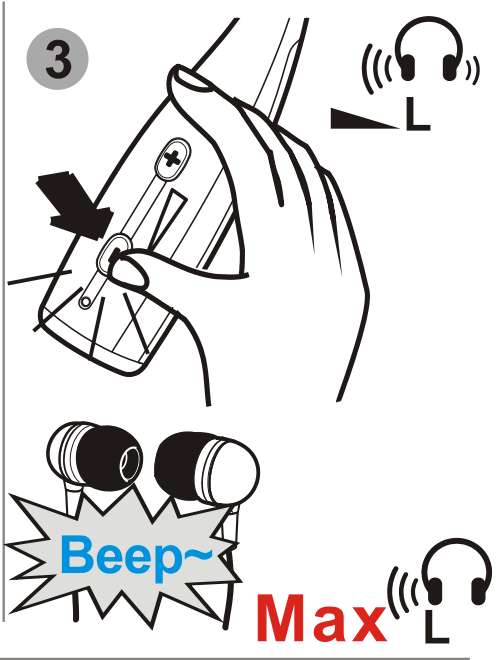
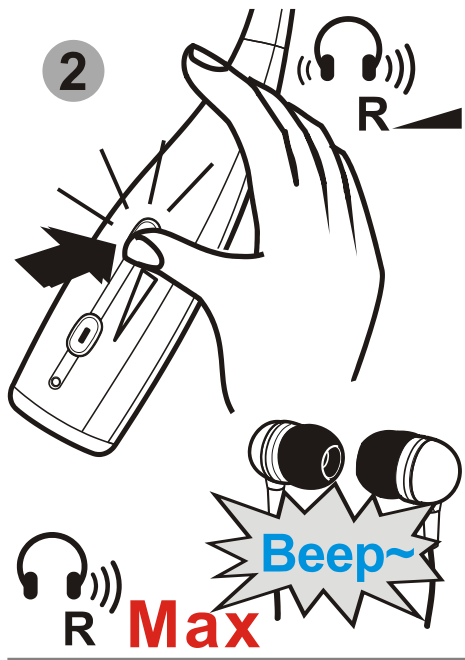
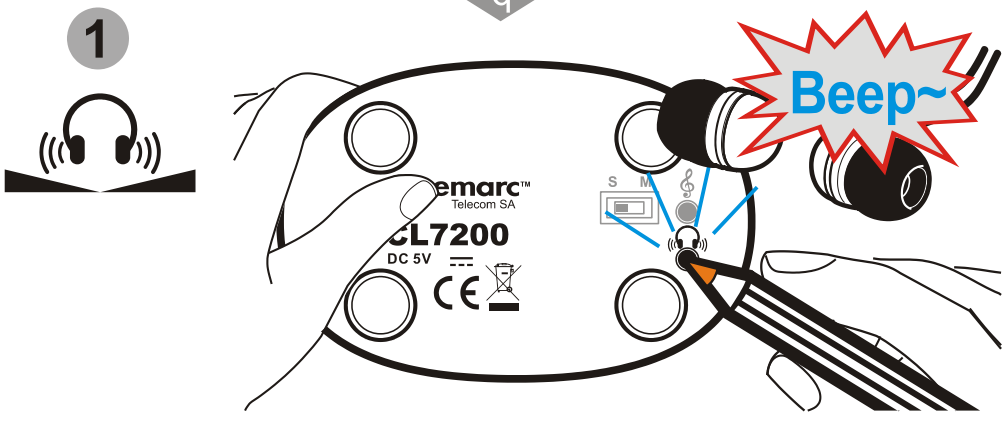
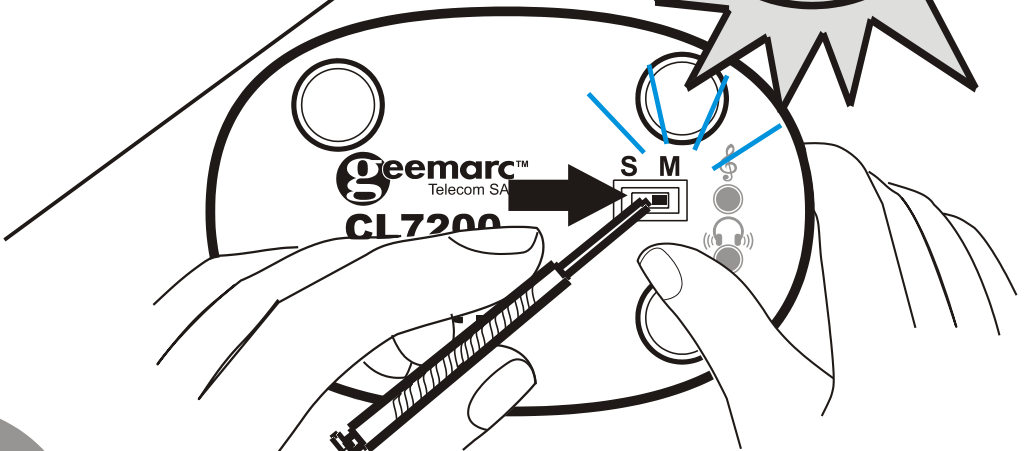
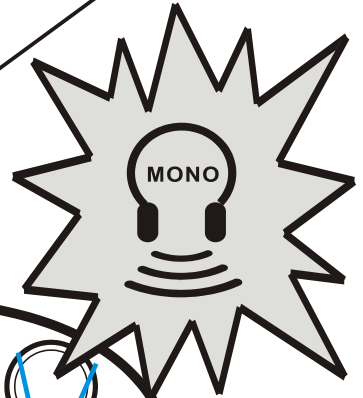
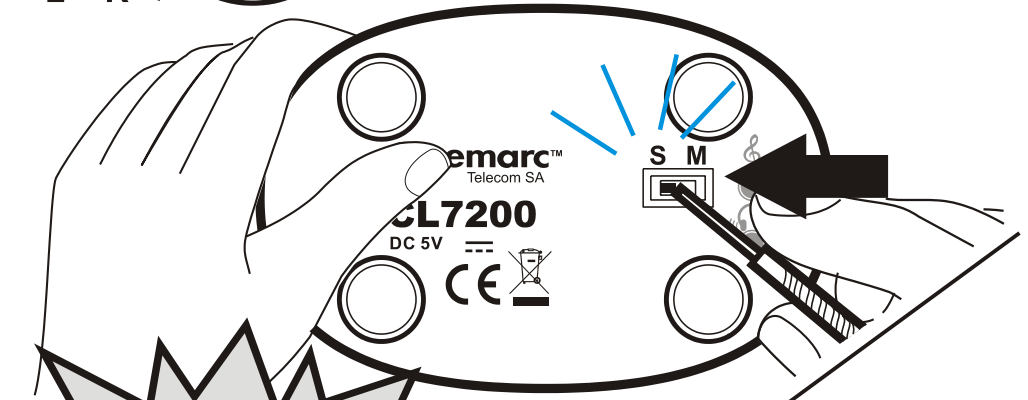
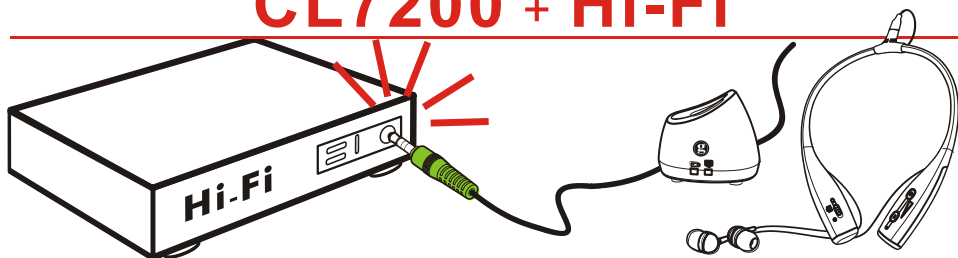
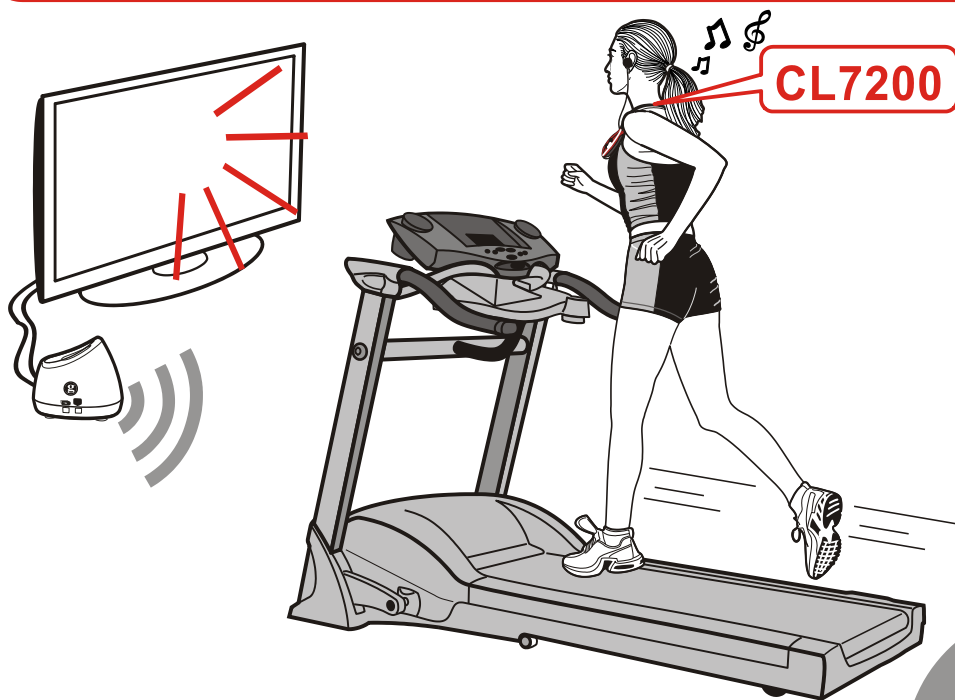
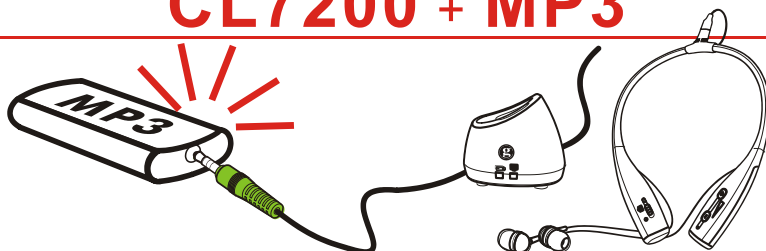
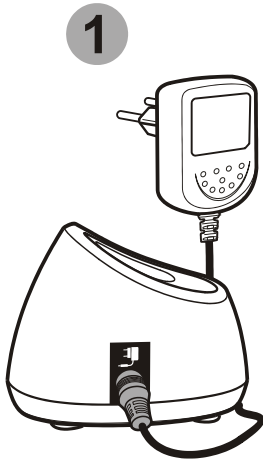
t

Extra charging pod

**En** Charging pod to allow for charging the CL7200 in a location convenient to you. Supplied with AC Adapter.

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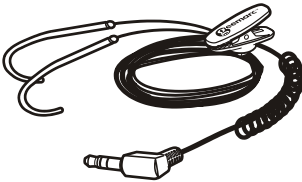
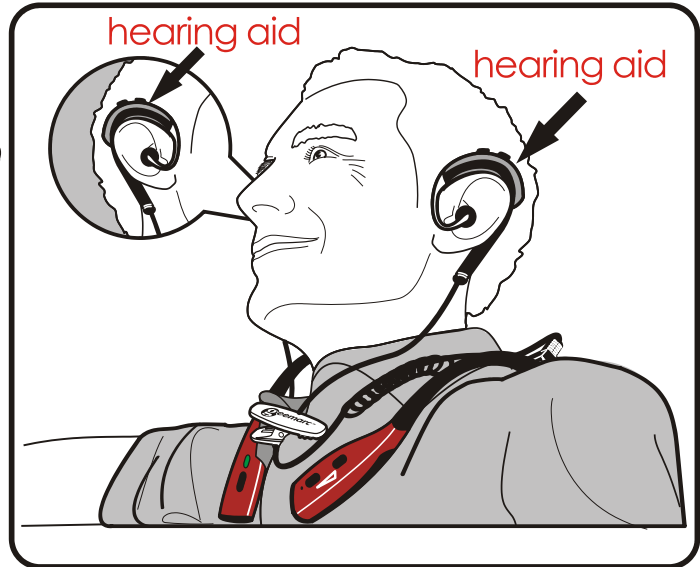
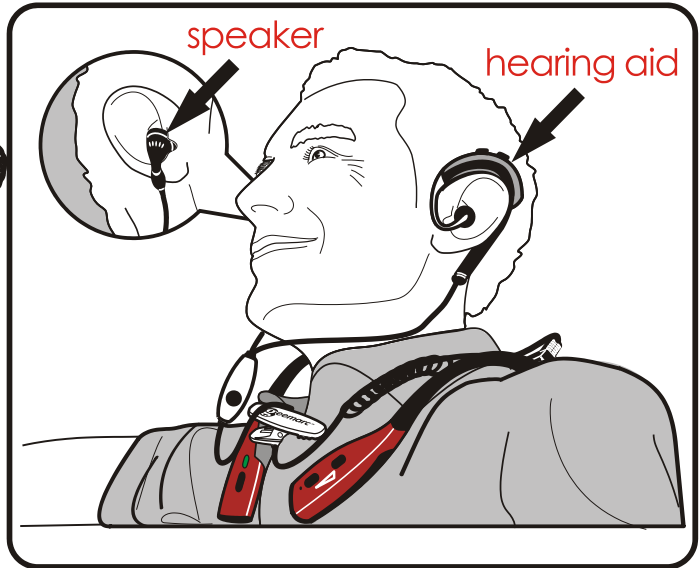
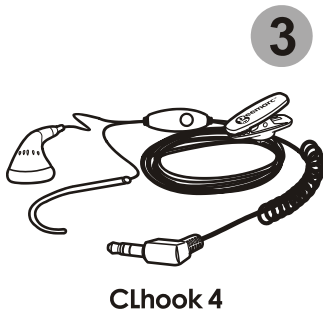
V

**En** Headset with 1 inductive ear hook and 1 standard speaker for users with 1 hearing aid.

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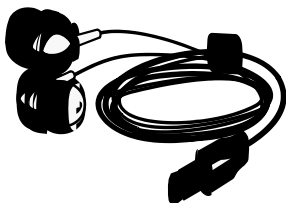
Extra headset - To be defined

## Safety & Care advice

To make the best use of this product and to use it safely, please read these instructions thoroughly before use and keep safe for future reference.

**Warning:** This product can be very loud. Take care to prevent others from using this product, if they do not have the same level of hearing loss. Excessive sound pressure from earphones and headphones can cause hearing loss.

**Caution: Electric shock. DO NOT** insert unattached plug heads in to mains power sockets. Plug heads **MUST** be fitted to the power adaptor.

**Electrical connection:** The unit is designed to operate from a 100-240VAC - 50/60Hz supply. (Classified as 'hazardous voltage' according to EN 60950 standard).

The unit does not incorporate an integral power on/off switch. To disconnect the power, either switch off supply at the mains power socket or unplug the AC adaptor. When installing the unit, ensure that the mains power socket is readily accessible.

**Caution: Risk of explosion if battery is replaced by an incorrect type. Dispose of used batteries according to the instructions.**

Keep the unit away from heat sources.

**DO NOT** expose the unit to direct sunlight.

Clean the Headset and ear pads regularly.

**DO NOT** modify the unit in any way.

**Please note:** The unit may be subject to interference from appliances that generate strong electrical or magnetic fields e.g. microwaves, mobile phones, lighting transformers etc.

## Battery Warnings:

- **DO NOT** attempt to dismantle.
- **DO NOT** dispose of in fire.
- **DO NOT** swallow.
- Keep away from children.
- Batteries are to be inserted with the correct polarity.
- **DO NOT** short-circuit the supply terminals.
- Exhausted batteries are to be removed from the product.
- Only use Geemarc supplied **2/3AAA** batteries.
- **Caution:** Risk of explosion if battery is replaced incorrectly or fitted with the wrong type of battery. Dispose of used batteries according to the instructions.

## General information

### Specifications

- Digital clear sound with 44.1 KSPS, 16 bit & 4Mbps RF link
- Reception adjustment up to 48dB
- 2.4 GHZ ISM band solution
- Frequency response : 40 - 15000Hz
- Up to 2 hour charge. Headset recharges itself when paced back on top of transmitter
- Operates up to 15 meter indoor from the transmitter
- working continuous time : more then 5 hours
- 32Ω(ohms) earphones
- 3.5 mm audio input stereo jack in transmitter with different adaptors to RCA or SCART connectors

### Disposal information

**Disposal of old electrical & electronic equipment (applicable in the European Union and other European countries with separate collection systems)**

The symbol on the product or its packaging indicates that this product may not be treated as household waste. Instead it should be handed over to the applicable collection point for the recycling of electrical and electronic equipment.

By ensuring this product is disposed of correctly, you will help prevent potential negative consequences for the environment and human health, which could otherwise be caused by inappropriate waste handling of this product.

For more detailed information about recycling this product, please contact your local council office, your household waste disposal service or the shop where the product was purchased.

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