91327

Nederlands

# My Ortho Cane

## INSTRUCTIES

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# My Ortho Cane

## MANUAL

- Remove the walking stick and insert the tubes into the appropriate openings.
- Insert the stick into the double handle. Unscrew the protective ring and secure the rubber plug.
- The My Ortho Cane has 4 different height settings, from 95 to 102 cm.
- The My Ortho Cane has an LED flashlight that runs with two CR2032 batteries.
- The My Ortho Cane has compartments in the handle. Place the strap around your wrist for more safety or use it to securely store the walking stick.
- When you stop to rest for a short time, bring the handles together at a 180 ° angle. For comfortable running, place them in a parallel position.

## WARNING

Do not mix new and used batteries. Remove the battery if you are not using the unit for an extended period of time. Do not expose the batteries to high temperatures or direct sunlight. Do not try to open it. Do not dispose of batteries in fire. Risk of explosion. Clean the device with a slightly moistened cloth. Do not use abrasive substances. The device must not be immersed in liquids. Do not touch with wet hands. Do not use in the vicinity of a water source or in damp places. This appliance is intended for private use only. Use the device only for the functions provided for it. Store the device in a dry and dust-proof place. Do not leave the unit unattended when switched on. Please do not attempt any repairs. This device is not intended for use by children or persons with physical or mental impairments, without supervision of an adult responsible for safety. Only suitable for domestic use. Dispose of disposable equipment in an environmentally friendly manner.

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**Kundenservice / Importeur:** HSP HanseShopping GmbH, Im Hegen 1, 22113 Oststeinbek, Deutschland
**Customer service / Importer:** HSP HanseShopping GmbH, Im Hegen 1, 22113 Oststeinbek, Germany
**Service / Importateur:** HSP HanseShopping GmbH, Im Hegen 1, 22113 Oststeinbek, Allemagne